

yellowRIGID TAPE

SPORTS TAPE, LINE GLUE COATING - INSTRUCTION FOR USE

DESCRIPTION

- Non-elastic, breathable tape made of cotton
- Acrylic, hypoallergenic coating
- Line glue coating together with alternate arrangement of the stripes covered with glue and those without it form grooves that allow free sweat and water flow, which reduces the tape's sensitivity and increases its resistance to environmental conditions
- Resistant to stretching
- Serrated edges - no need for scissors
- In case of big application or sensitive skin it is recommended to use foam underwrap
- Available colours: white
- Available sizes: 2.5 cm x 9.1m; 3.8 cm x 9.1m; 5cm x 9.1m
- Single use
- Non-sterile

INDICATION

- For athletic taping
- Sports injuries prevention
- Protecting the joints from excessive mobility after the injury, which allows for activity continuation
- Protection of the areas exposed to abrasions



INSTRUCTION FOR USE

- Choose the width of tape adequately to the place of application.
- If the area is sensitive, hairy or the epidermis is exposed damages, it is recommended to secure it with an underwrap.
- If the underwrap is not being used it is recommended to wipe the skin with an alcoholic pad in order to degrease it.
- If the skin had been degreased it should be properly dried before taping.
- Take the tape out of the packaging.
- Untuck the tape from the roll and tear or cut it into the desired length.
- Put the tape on the place of application, directly on skin or on the underwrap.
- The taping should be performed by using pieces of tape, not by wrapping the tape around the limb.
- Check the application for any signs of ischemia.
- If the tapes peels off or the adhesive layer is contaminated it is necessary to change it because it loses its stabilization properties.
- The tape should be removed after the activity during which it was used is finished.
- To remove the application, each element should be peeled off. It is also possible to cut the whole application with scissors, paying attention in order not to damage the skin.

REMARKS AND WARNINGS

- It is recommended to consult a specialist before first use.
- For single use only. Repeated use may impact the health or life.
- Non-sterile, do not use directly on wounds or damaged epidermis.
- Do not use after the expiry date specified on the packaging.
- Do not use if the packaging is damaged.
- The product should be used in accordance with aseptic principles.
- In case of any signs of allergy such as: irritation, itching or rash, the tape should be removed. Further use is not recommended.
- In case of any signs of ischemia, remove the application and reperform it with less tightness.
- Store in dry rooms with a stable temperature within range from +5 to +30 °C and humidity between 30% and 80%. Protect from sunlight and humidity.

REF	SIZE	COLOUR	INDIVIDUAL PACKAGING	BULK PACKAGING
YRT-2591-W	2.5cmx9.1m	white	12 pcs	30 x 12 pcs
YRT-3891-W	3.8cmx9.1m	white	6 pcs	30 x 6 pcs
YRT-591-W	5cmx9.1m	white	6 pcs	30 x 6 pcs

EXPLANATION OF SYMBOLS USED ON THE PACKAGING

	name and address of the Manufacturer
	CE mark
	batch code specifying the production serial number
	product catalogue number
	non-sterile
	single use
	expiration date in format: RRRR-MM-DD
	latex-free
	keep away from sunlight
	keep away from humidity
	storage temperature range
	storage humidity range
	packaging suitable for recycling